

Company Statement on Zoom' updated privacy.

On August 11th, 2023; Zoom Video Communications Inc. updated their Privacy Statement (<https://explore.zoom.us/en/privacy/>)

“This Privacy Statement describes the personal data we collect and/or process (which may include collecting, organizing, structuring, storing, using, or disclosing) to provide products and services offered directly by Zoom Video Communications, Inc. (“Zoom”), including Zoom’s websites, its meetings, webinars, and messaging platform, related collaborative features, and Zoom App Marketplace (“Zoom products and services” or “products and services”)...”

UpCare Partners and Associates Inc. is formerly making the statement that the company do not agree nor allow Zoom Video Communications Inc. to record, collect, retain, store, use or disclose any of the content being discussed and share today on this call.



A Truly Portable Brain MRI Scanner

A game changer for both point-of-care and remote communities



UpCare

Agenda



Introduction of the technology
of portable MRI



Applications in Canada




Cost Analysis Insights from
"Feasibility & Cost Analysis of
Portable MRI conducted in a
Remote Setting in Canada
(2023)"



Application of ultra-low field
portable MRI in the ICU to
improve CT and MRI access



Q&A session

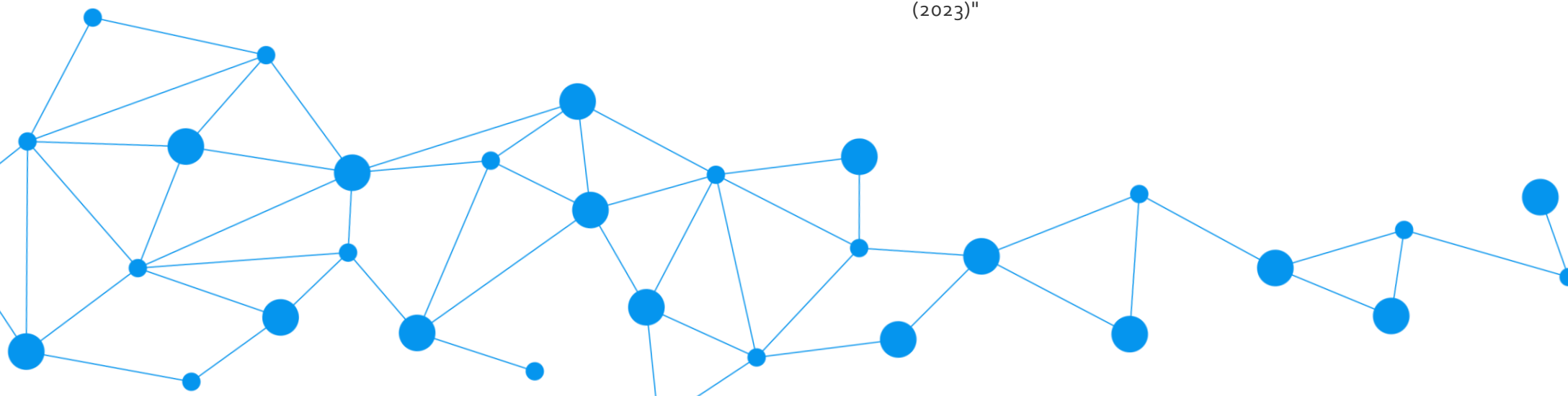

HYPERFINE

World First Mobile MR Imaging




*By Elena Klimina t.i.m. BComm
UpCare Product and Application Specialist*




Specs

- 0.064 Tesla
- 33" wide x 59" high
- 15 Amp wall outlet
- ≈ 30 minutes user training

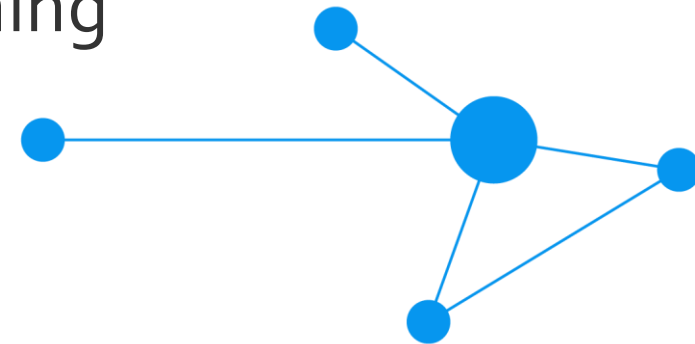

Available Sequences:


- 3D T2-weighted
- 3D T1-weighted
- 3D Flair
- DWI + ADC

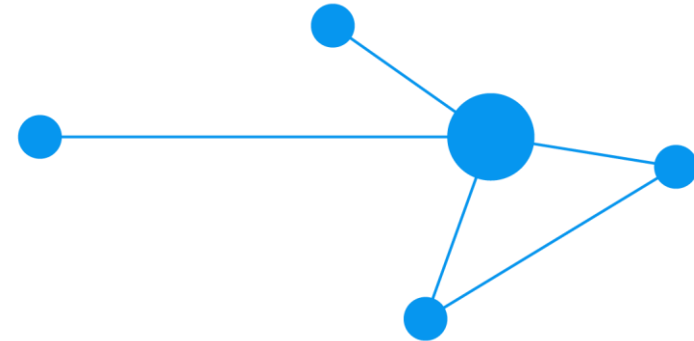



Truly Portable Brain MRI Scanner

- Can be safely transported in the hallways
- Does not require any infrastructural changes
- Images can be read remotely in real-time

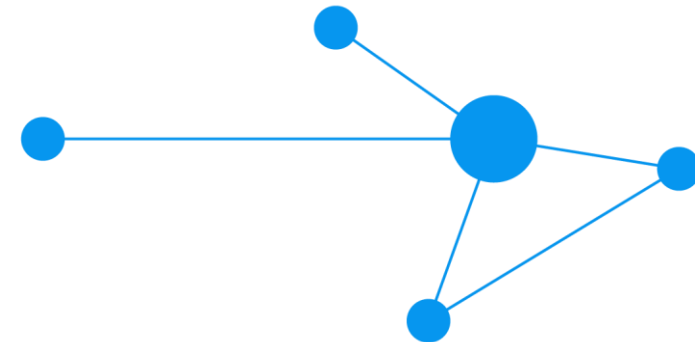



Hyperfine point-of-care MRI, Swoop™, can go anywhere.

Small and highly portable, Swoop™ can be used in:

- ER
- ICU
- Stroke Unit
- Ambulatory care clinics
- Pediatric hospitals
- Or anywhere else you can imagine



Hyperfine point-of-care MRI, Swoop™

Used in the clinical diagnosis and care for the following neurological conditions:

- Acute mental status change
- Head Injury
- Stroke recovery
- Hydrocephalus (diagnosis and monitoring of shunt)